

NEW



FIRST AID STATION

& guidance for use



EYEWASH KIT

EYEWASH KIT CONTENTS

- STEROWASH 500ml
- EYEPAD DRESSINGS
- SHATTERPROOF MIRROR
- STEROWASH 20ml POPS

EMERGENCY

Emergency Eye wash Station

FIRST AID KIT

FIRST AID KIT CONTENTS

- FOL BLANKET
- SCISSORS
- GLOVES
- SAFETY PIN
- MICROPOROUS TAPE
- LEAFLET
- BURN DRESSINGS
- WOUND DRESSINGS
- SALINE WIPES
- REINFORCE FACE SHIELD
- BANDAGES

UPDATED TO NEW BS5599-1:2019 STANDARD

How to use the EYEWASH KIT in an emergency

STEP 1:

Twist off the top of the pod or bottle to open the eyewash solution provided.

STEP 2:

Open eyes wide and position the open bottle of solution above the eye. Use the mirror provided if using without assistance.

STEP 3:

Gently squeeze the bottle allowing the liquid to flow freely into and across the eyeball.

STEP 4:

Use the mirror provided to observe the eye. Seek further medical assistance if necessary and use eyepad dressings to protect the eye.

STEP 5:

Discard of the bottle after use. **DO NOT REUSE.** Replace any used contents of the kit for use in the next emergency.

1m

1m

A brand new and improved first aid station

The first 10 seconds after the eye has been exposed to a hazardous substance are critical, especially when that substance is corrosive.

- Eyewash stations must be fixed to a wall near to the potential hazard.
- E.g. next to a fork lift charging point as battery acid splashes could cause blindness.
- Eyewash should be wall mounted where there are toxic, corrosive, acidic, or caustic, chemicals.
- Perfect to remove sand and dust particles which can damage the eye if not removed quickly.

If you want a station made to your exact requirements with any combination of first aid kits, please call us on 0161 9022 3030

First Aid Station

Save money on expensive first aid stations by utilising the super strong brackets from Steroplast.

You can create a "First Aid Station" utilising the brackets as demonstrated in this picture.

You can have any combination of kits, based on whatever suits your workplace, so the choice is yours.

Position the kits vertically if it helps where you have less space.

Keeps all your first aid needs in plain sight, you'll always know where they are.



Essential part of a first aid station

If your staff are split over more than one floor, you must have at least one kit per floor. First aid kits should be **positioned where people see them every day**, so that in the event of an emergency, people will remember where to find them.



First aid kit compliance made nice and easy by Steroplast Healthcare